Therabody

RecoveryAir JetBoots

Quick Start Guide

Portable Recovery.

Table of Contents

EN	1-4
DE	5-8
FR	9-12
IT	13-16
ES	17-20
NL	21-24
PL	25-28
EL	29-32
NO	33-36
SR	37-40
SV	41-44
TR	45-48
PT	49-52
RU	53-56
AR	57-60
JP	61-64
KR	65-68
TW	69-72
CN	73-76

RecoveryAir JetBoots

Integrated Pneumatic Compression System

Fully wireless compression boots.

Your RecoveryAir JetBoots come with two compression boots, each with integrated pumps. Plus a 36W charger, a splitter charging cord, and a carrying pouch.

Getting started

1. To get started, put your boots on and get into a comfortable position.
2. Once ready, turn ON the RecoveryAir JetBoots integrated pump, by pressing the power button.
3. If your boots are not automatically paired, press the pressure and time buttons at the same time and hold both for 5 seconds to enable the pairing of the two boots.
4. The pressure and time settings can be adjusted directly from the integrated pumps before your treatment begins. More settings can be made from the Therabody app.
5. Use your JetBoots at the same time as a pair or independently of each other.

Prepare your treatment

1. After turning on and pairing your RecoveryAir JetBoots, you are ready to choose your program settings.
2. Access two preset program options when connected to the Therabody app: Recover and Warm-up.
3. Once you have chosen your desired program, use the Pressure button to adjust the pressure setting.
4. Then, toggle through the time options using the Time button, to set your desired treatment time (20min, 40min, 60min, or continuous).
5. Once you have selected the treatment options that are right for you, press start, relax, and you'll be on your way to recovery in no time.

Feature Callouts

A Buttons

Power Button

*Start/Pause/
Stop Button*

Time Button

4 time intervals (20, 40, 60, and continuous)

Pressure Button

Adjustable Pressure Range (25-100 mmHg) in increments of 25 mmHg. Adjustable pressure changes via the Therabody app in increments of 5 mmHg

B Indicators

Warning Indicator

Indicator lights up solid ON or blinking when a device malfunction is detected

Bluetooth Indicator

Indicator turns ON only when connected to the App

Battery Indicator

3 fully lit bars to indicate battery life

Linked Indicators

No indicator: Boot has not been pre-paired

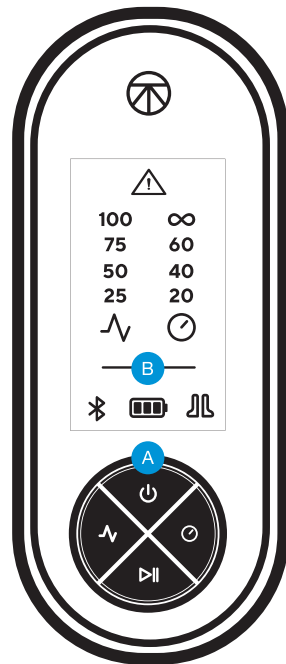
1 Linked Indicator:

Boot is pre-paired but was not able to link/connect with the other boot

2 Linked Indicator: Boots are pre-paired and linked/connected with the other boot

Smart Features

The RecoveryAir JetBoots also connect to the Therabody app via Bluetooth, giving you control of your session from the convenience of your smartphone.

Not only can you easily access the Recover and Warm-up treatment options, but you can also adjust the pressure and time.

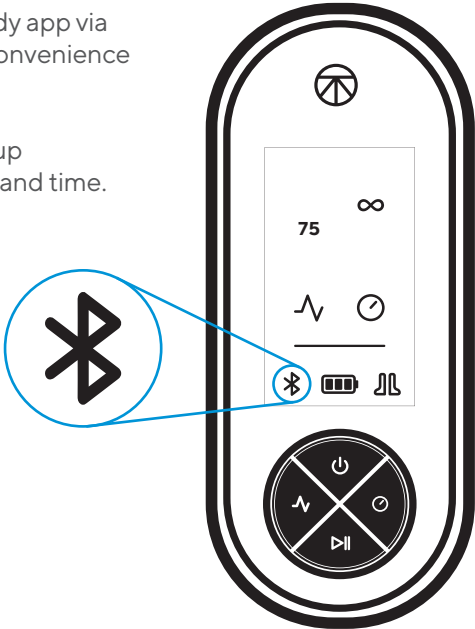
Scan the QR code to get started.

Safety Warnings and Precautions

- 1. Do not stand or walk while wearing the garment.
- 2. See User Manual for all safety warnings and precautions.

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