Therabody

RecoveryTherm™

Hot Vibration Back & Core

Unit Warnings

Table of contents

EN	3-8
FR	9-13
ES	14-18
IT	19-23
DE	24-29
CN	30-34

RecoveryTherm™ Hot Vibration Back & Core Warnings and Guidance (Precautions and Contraindications)

Background

This garment is intended to be used on the back. If you have any specific medical conditions or concerns, please consult your physician before using this product. At times, it will be advisable to modify how garments are used (precautions) and times when it is not appropriate to use certain garments (contraindications). The following document highlights these for the RecoveryTherm™ Hot Vibration Back & Core as of the print date. For up-to-date information, please visit us online at www.therabody.com.

Important Safety Information

General RecoveryTherm™ Hot Vibration Back & Core Use

Read the full Warnings and Guidance prior to using the RecoveryTherm™ Hot Vibration Back & Core.

This garment is intended for use by people in good health. This garment is contraindicated against and should not be used by those who are or may be pregnant. It is also contraindicated for those with acute or severe cardiac, liver, or kidney disease/condition or other underlying conditions linked to these diseases. It is not recommended that this garment is worn over various skin infections and conditions, or directly over a surgical site/hardware, bone fracture, or if you are experiencing unexplained pain in the area. It is also advised that this garment is not used for those who have neurological and/or peripheral conditions as well as other conditions that may influence or alter sensation and perception. In the event of any adverse reaction, discontinue use. Please consult your physician prior to use if you are unsure if you have any of the previously listed conditions. **Immediately stop using the garment at the first sign of discomfort.**

FN Adult supervision should be provided for those under the age of 18 using this garment. If you have any medical concerns regarding the use of this garment, please consult your physician prior to using the garment.

RecoveryTherm™ Hot Vibration Back & Core Safety, Precautions, and Contraindications

These recommendations are derived from consultation with medical experts and the published research regarding precautions and contraindications for heat and vibration therapy. For up-to-date information, please visit us online.

Precautions:

Due care is required in these circumstances, and the use of the garment may need to be modified. Where appropriate, or if you have any concerns, seek the advice of a medical professional.

- Recent injury or surgery
- Hypertension (controlled)
- Abnormal sensations (e.g., numbness)
- Sensitivity to heat or pressure
- Mild peripheral neuropathy
- This garment should fit tightly but should NOT restrict circulation in any way
- Do not wear this garment overnight while sleeping

Contraindications:

The following are circumstances where the potential risks may outweigh the benefits. Consult a medical professional before use.

- Skin rash, open wounds, blisters, local tissue inflammation, infections, bruises, or tumors
- Herniated disks
- Bone fracture or myositis ossifications
- Hypertension (uncontrolled)
- Vascular/arterial and circulation diseases, including blood clots, phlebitis, thromboses, and varicose veins
- Acute or severe cardiac, liver, or kidney disease
- Severe peripheral neuropathy or other causes of sensory impairment
- Bleeding disorders
- Connective tissue disorders
- Medications that may thin the blood or alter sensations
- Direct pressure over surgical site or hardware
- Pacemaker or ICD
- Implanted devices such as IUDs, metal pins/plates
- History of embolism
- Allergy to garment material
- Do not use in the presence of unexplained pain

**ADDITIONAL UNIT WARNINGS
LIMITED WARRANTY**

For full warranty information, please visit www.therabody.com/warranty. To request a copy of the warranty by mail, you may send a request to the following address:

Therabody - Warranty
Attn: Customer Service
6100 Wilshire Blvd., Suite 200
Los Angeles, CA 90048

Please note, this is not a return address or a retail location. No products or packages will be accepted at this location.

UNIT WARNINGS

BEFORE USING OR CHARGING THE RecoveryTherm™ Hot Vibration Back & Core, READ ALL INSTRUCTIONS AND CAUTIONARY MARKINGS IN THIS MANUAL, ON THE CHARGER, ON THE BATTERY, AND THE RecoveryTherm™ Hot Vibration Back & Core. If your garment doesn't turn on or the battery indicator displays a low battery level, please charge before first use. The RecoveryTherm™ Hot Vibration Back & Core is intended for commercial or home use.

This RecoveryTherm™ Hot Vibration Back & Core is not intended to diagnose diseases. Therabody strives to make the RecoveryTherm™ Hot Vibration Back & Core as safe for intended use as possible. This is an advanced mechanical tool with electric components. If the RecoveryTherm™ Hot Vibration Back & Core and its accessories are not used or maintained properly, there is a risk of fire, electric shock, or injury. When using the RecoveryTherm™ Hot Vibration Back & Core, the following basic precautions should always be adhered to:

1. **USE ONLY AS INSTRUCTED.** Use the RecoveryTherm™ Hot Vibration Back & Core as described in the RecoveryTherm™ Hot Vibration Back & Core Instruction Manual only. Use only Therabody recommended accessories and replacement parts. Do not carry out any maintenance other than as advised by Therabody.
2. **NOT FOR CHILDREN.** The RecoveryTherm™ Hot Vibration Back & Core and charger are not intended for use by young children or persons with reduced physical, sensory, or reasoning capabilities or lack of experience and knowledge unless a responsible adult has given them supervision or instruction. RecoveryTherm™ Hot Vibration Back & Core is not to be used as a toy. Do not play with, bend, or pull the electrical components. Advise children not to play with the RecoveryTherm™ Hot Vibration Back & Core or charger. Adult supervision should be provided for those under 18 using this garment.
3. **USAGE.** Discontinue the use of the RecoveryTherm™ Hot Vibration Back & Core if you are feeling severe pain. Do not use the RecoveryTherm™ Hot Vibration Back & Core garment on the face or genitals. If you feel pain or discomfort beyond what's expected from a hot shower at any point during the hot therapy treatment, stop the treatment immediately and remove the garment.
4. **CHARGING LOCATIONS.** Charge the RecoveryTherm™ Hot Vibration Back & Core with a USB-C charger. The RecoveryTherm™ Hot Vibration Back & Core should be charged indoors in a well-ventilated, dry location. Do not charge the RecoveryTherm™ Hot Vibration Back & Core outdoors, in a bathroom, or within 10 feet (3.1 meters) of a bathtub or pool. Do not use the RecoveryTherm™ Hot Vibration Back & Core or charger on wet surfaces, and do not expose the charger to moisture, rain, or snow. Do not use the RecoveryTherm™ Hot Vibration Back & Core or its compatible charger in the presence of explosive atmospheres (gaseous fumes, dust, or flammable materials). Sparks may be generated, possibly causing a fire.

5. **UNPLUGGING THE CHARGER.** Pull the plug, not the cord, to reduce the risk of damage to the electrical plug and cord. Never carry the charger by its cord. Keep the cord from heat, oil, and sharp edges. Do not use or charge with a damaged cord. Do not stretch the charger cord or place the cord under strain. Keep the cord away from heated surfaces. Do not handle the charger, including the charger terminals, or RecoveryTherm™ Hot Vibration Back & Core with wet hands. Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
6. **THERABODY USB-C ADAPTER CARE.** Unplug the adapter when not in use. Store cord to ensure it is not stepped on, tripped over, or otherwise subjected to damage or stress. Replace a damaged charger. Keep the cord away from heated surfaces. Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. For long-term storage, store with a fully charged battery. Therabody is not responsible for damages that may occur due to the use of third-party chargers.
7. **DO NOT OPERATE UNDER BLANKET AND PILLOW.** Excessive heating can occur and cause fire, electric shock, or injury.
8. **BATTERY.** There is only one correct battery insertion position in the battery dock. Do not force the battery into place. The Therabody logo should be facing upright when inserted.
9. **DISPOSING OF BATTERIES.** Always dispose of RecoveryTherm™ Hot Vibration Back & Core batteries according to federal, state, and local regulations. Contact a recycling agency in your area for recycling locations. Even discharged batteries contain some energy. Before disposing, use electrical tape to cover the RecoveryTherm™ Hot Vibration Back & Core 16V connector/terminals to prevent the battery from shorting, which could cause a fire or explosion.
10. **STORING THE RecoveryTherm™ Hot Vibration Back & Core, BATTERY, AND USB-C ADAPTER.** Store in a cool, dry place. Only charge the RecoveryTherm™ Hot Vibration Back & Core when the ambient temperature is between 0°C/32°F and 40°C/104°F. Do not store the RecoveryTherm™ Hot Vibration Back & Core, batteries, or charger where temperatures may exceed 40°C/104°F, such as in direct sunlight, in a vehicle, or in a metal building during the summer.
11. **GARMENT CARE.** The RecoveryTherm™ Hot Vibration Back & Core is NOT waterproof or machine washable. Do not allow excess liquids or water to enter the product. Remove the battery before cleaning. To clean, wipe the inner surface of the strap with a dampened microfiber cloth. Do not overbend the strap. For traveling or storage, gently fold the strap into three sections.
12. **GARMENT CARE.** The garment has movable pieces on the inner side that adapt to your body to provide a comfortable and effective thermal treatment. Do not pull on or disassemble movable pieces, as this can impact the efficiency and performance of the garment.

SAVE THESE INSTRUCTIONS

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause interference. (2) This device must accept any interference, including interference that may cause undesired operation of the device. This equipment complies with IC RSS-102 radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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