Therabody

SmartGoggles™

Unit Warnings

Table of contents

| | |
|------|---------|
| EN | 3 - 7 |
| ES | 8 - 12 |
| FR | 13 - 17 |
| IT | 18 - 22 |
| DE | 23 - 27 |
| ESMX | 28-32 |
| FRCA | 33-37 |
| SC | 38-42 |
| TC | 43-47 |
| KR | 48-52 |
| JP | 53-57 |

SmartGoggles™ Warnings and Guidance (Precautions and Contraindications)

Background

This device is intended for use by people in good health. If you have any specific medical conditions or concerns, please consult your physician before using this product. At times, it will be advisable to modify how devices are used (precautions) and times when it is not appropriate to use certain devices (contraindications). The following document highlights these for SmartGoggles as of the print date. For up-to-date information, please visit us online at www.Therabody.com.

Important Safety Information

General SmartGoggles™ Use

Read the full Warnings and Guidance prior to using SmartGoggles™ Device.

SmartGoggles should not be used while in motion or performing activities like walking, driving, or operating machinery. This device is intended for use by people in good health. This device is contraindicated against and should not be used by those with acute or severe cardiac, liver, or kidney disease/condition or other underlying conditions linked to these diseases. It is not recommended that this device is worn over various skin infections and conditions, or directly over a surgical site/hardware, bone fracture, or if you are experiencing unexplained pain in the area. It is also advised that this device is not used for those who have neurological and/or peripheral conditions as well as other conditions that may influence or alter sensation and perception. In the event of any adverse reaction, discontinue use. Please consult your physician prior to use if you are unsure if you have any of the previously listed conditions. ***Immediately stop using the garment at the first sign of discomfort.***

Adult supervision should be provided for those under 18 using this device. If you have any medical concerns regarding using this device, please consult your physician before using the device.

SmartGoggles™ Safety, Precautions, and Contraindications

These recommendations are derived from consultation with medical experts and the published research regarding precautions and contraindications for myofascial, heat, and vibration therapy. For up-to-date information, please visit us online.

Precautions:

Due care is required in these circumstances, and the devices' use may need to be modified. Where appropriate, or if you have any concerns, seek the advice of a medical professional.

- Recent injury or surgery
- Hypertension (controlled)
- Abnormal sensations (e.g., numbness)
- Sensitivity to heat or pressure
- Bony prominences or regions
- Mild peripheral neuropathy
- Remove eye lenses (i.e. contact lenses) before using the device
- This device should fit tightly but should NOT restrict circulation in any way

Contraindications:

The following are circumstances where the potential risks may outweigh the benefits. Consult a medical professional before use.

- Skin rash, open wounds, blisters, local tissue inflammation, infections, bruises, or tumors
- Hypertension (uncontrolled)
- Vascular/arterial and circulation diseases, including blood clots, phlebitis, thromboses, and varicose veins
- Acute or severe cardiac, liver, or kidney disease
- Severe peripheral neuropathy or other causes of sensory impairment
- Bleeding disorders
- Connective tissue disorders
- Medications that may thin the blood or alter sensations
- Direct pressure over surgical site or hardware
- Pacemaker or ICD
- History of embolism
- Do not use in the presence of unexplained pain

ADDITIONAL UNIT WARNINGS LIMITED WARRANTY

For full warranty information, please visit www.therabody.com/warranty. To request a copy of the warranty by mail, you may send a request to the following address:

Therabody - Warranty
Attn: Customer Service
6100 Wilshire Blvd., Suite 200
Los Angeles, CA 90048

Please note, this is not a return address or a retail location. No products or packages will be accepted at this location.

Distributed by: Therabody Australia Pty. Ltd., The Commons South Yarra, 11 Wilson Street, South Yarra VIC 3141
australia@therabodycorp.com

UNIT WARNINGS

BEFORE USING OR CHARGING SmartGoggles, READ ALL INSTRUCTIONS AND CAUTIONARY MARKINGS IN THIS MANUAL, ON THE CHARGER, ON THE BATTERY, AND SmartGoggles. If your device doesn't turn on or the battery indicator displays a low battery level, please charge before first use. SmartGoggles are intended for commercial or home use.

SmartGoggles are not intended to diagnose, mitigate, or treat diseases. Therabody strives to make SmartGoggles as safe for intended use as possible. SmartGoggles is an advanced mechanical tool with electric components. If SmartGoggles and their accessories are not used or maintained properly, there is a risk of fire, electric shock, or injury. When using SmartGoggles, the following basic precautions should always be adhered to:

1. **USE ONLY AS INSTRUCTED.** Use SmartGoggles as described in the SmartGoggles Instruction Manual only. Use only Therabody recommended accessories and replacement parts. Do not carry out any maintenance other than as advised by Therabody.
2. **NOT FOR CHILDREN.** SmartGoggles and charger are not intended for use by young children or persons with reduced physical, sensory, or reasoning capabilities or lack of experience and knowledge unless a responsible adult has given them supervision or instruction. SmartGoggles are not to be used as a toy. Advise children not to play with the SmartGoggles or charger. Adult supervision should be provided for those under 18 while using this device.
3. **CHARGING LOCATIONS.** Ensure that the battery status LED is OFF before charging. Insert the USB-C cable into the port (behind the Power Button) to charge SmartGoggles. Do not use SmartGoggles while charging. Avoid skin contact with the connector when the USB-C to USB-A Cable is plugged into a power source as it may cause discomfort or injury. Do not sleep or sit on the USB-C connector. Charge SmartGoggles with a USB-C charger. SmartGoggles should be charged indoors in a well-ventilated, dry location. Do not charge SmartGoggles outdoors, in a bathroom, or within 10 feet (3.1 meters) of a bathtub or pool. Do not use SmartGoggles or charger on wet surfaces, and do not expose the charger to moisture, rain, or snow. Do not use SmartGoggles or its compatible charger in the presence of explosive atmospheres (gaseous fumes, dust, grain, metal powders, or flammable materials). Sparks may be generated, possibly causing a fire. Obey all signs and instructions.
4. **UNPLUGGING THE CHARGER.** Do not overcharge SmartGoggles device. Pull the plug, not the cord, to reduce the risk of damage to the electrical plug and cord. Never carry the charger by its cord. Keep the cord away from heat, oil, and sharp edges. Do not use or charge with a damaged cord. Do not stretch the charger cord or place the cord under strain. Keep the cord away

from heated surfaces. Do not handle the charger, terminals, or SmartGoggles with wet hands. Always unplug this appliance from the electrical outlet immediately after using and before cleaning.

5. **THERABODY USB-C CHARGING CABLE CARE.** Unplug the adapter when not in use. Store cord away so that it will not be stepped on, tripped over, or otherwise subjected to damage or stress. Replace a damaged charger. Keep the cord away from heated surfaces. Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. For long-term storage, keep the battery fully charged. Therabody is not responsible for damages that may occur due to the use of third-party chargers.
6. **USAGE.** Discontinue the use of SmartGoggles if you are feeling severe pain. Adjust the strap to fit comfortably on your head. The Therabody SmartGoggles should perfectly align with your temples and sit comfortably on your nose bridge. Always adjust SmartGoggles while the device is OFF. Long press the Power Button to turn off SmartGoggles. Make sure the Battery status LED is OFF. Use SmartGoggles on clean, dry skin. Remove makeup before SmartGoggles use.
7. **DEVICE CARE.** Remove eyeglasses and contact lenses before using SmartGoggles. Handle SmartGoggles with care. SmartGoggles contain electronic components that can be damaged if dropped, burned, punctured, or crushed. Do not disassemble the SmartGoggles or attempt to repair SmartGoggles yourself. Disassembling SmartGoggles may damage it, result in loss of water resistance, and may cause injury. If SmartGoggles is damaged or if malfunctions occur, contact Therabody Customer service. Do not use damaged SmartGoggles, for example, visible water intrusion or damaged strap, as these may cause injury. Avoid heavy exposure to dust or sand.
8. **CLEANING INSTRUCTIONS.** Clean SmartGoggles with a damp cloth if they come in contact with anything that may cause stains, or other damage, such as dirt or sand, makeup, ink, soap, detergent, acids or acidic foods, or with liquids other than water, including those that may lead to skin irritation such as: sweat, salt water, soapy water, pool water, perfume, insect repellent, lotions, sunscreen, oil, adhesive remover, hair dye, or solvents.

SAVE THESE INSTRUCTIONS

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause interference. (2) This device must accept any interference, including interference that may cause undesired operation of the device. This equipment complies with IC RSS-102 radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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